**DE**

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---

**FR**

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Similicuir et housses de matelas

---

**IT**

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## 4 ENGLISH

Dear customer

Thank you very much for choosing HESS Medizintechnik AG. You will enjoy this durable product. We attach great importance to our focus on quality, functionality and ergonomics. We guarantee you a careful selection of materials, the use of modern production technology and constant quality control. The products of HESS Medizintechnik AG comply with safety and hygiene standards and meet the high demands of everyday hospital and practice use at all times.

Follow the care instructions. In this way you can take full advantage of our product and at the same time prevent improper use.

Your HESS Medizintechnik AG

### 4.1 Cleaning

The following instructions for cleaning and disinfection are intended as a guide. Only use cleaning agents and disinfectants at temperatures between 5°C and 50°C. Follow the manufacturer's instructions for the dosage of the disinfectant. In any case, the disinfectant should first be applied to the surface in a suitable place to check compatibility with the material at the desired dosage and duration of application.

---

Important note!

Always follow the manufacturer's instructions for the cleaning agent or disinfectant used!

---

#### Cleaning the upholstery

The surfaces are extremely easy to clean. To maintain the suppleness and original lustre, please observe the following:

Clean regularly with lukewarm water, pH neutral soap and a soft brush. Do not use strong, abrasive or solvent-based cleaning agents, polishes, chemical cleaners or other products not recommended by us. Stains, especially from ballpoint pens, coffee, wine, blood or denim, should be cleaned immediately, as they may not be able to be removed later without leaving any residue. (Fresh biro stains can be removed with adhesive tape). For stubborn stains, we recommend our cleaning agent HESS-MultiClean® (Art. H06-991). Please refer to the description on the bottle label.

Cleaning process:

1. use a cloth with a lukewarm soap solution and wipe off any dirt with the cloth.
2. Use a separate, clean cloth with clear water and wipe off any soap residue.
3. Dry the upholstery with a cloth if necessary.

---

Important note

- Correct and regular cleaning extends the service life of the upholstery cover.
- Do not place any colouring, hot or sharp objects on the upholstery. This can damage the upholstery cover.
- The following products must be avoided at all costs: Pure alcohol / acetone, all solvents and detergents, abrasive care products, perchloroethylene, all types of wax and trichloroethylene.

**The guarantee only covers production defects.** Discolouration and embrittlement of the upholstery cover are excluded.

---

## 4.2 Disinfection

We recommend using the disinfectants marked in green from the disinfectant overview. Pay particular attention to the solvent concentration and proper handling in accordance with the manufacturer's instructions. Disinfectants can impair the service life of the upholstery cover if used incorrectly. Before using other disinfectants, ask the manufacturer of the disinfectant about the suitability of the product. The following work steps must be strictly adhered to.

### cleaning:

Depending on the type and concentration of the disinfectant, the coating of a polymer, soft coating swells more or less. For this reason, it is essential to clean the surface beforehand so that dirt particles are not trapped in the coating. Clean with lukewarm water, pH neutral soap and a soft brush or soft cotton cloth (not microfibre).

### disinfection:

All synthetic leather coatings are affected and altered in the long term by the use of disinfectants. The disinfectants tested and recognised by the Robert Koch Institute for surface and wipe disinfection (according to D-LAB standard, DIN EN ISO 2812-4) can be used under normal conditions for this artificial leather and the mattress covers.

The following points must be observed:

- We accept no responsibility if, between our assessment and the date of manufacture, the manufacturer has made changes to the formulation that cause damage and do not comply with our test.
- It is necessary to respect the instructions of the detergents indicated (concentration, duration, etc.)
- The use of detergents and disinfectants may cause discolouration, but the coating will not be affected
- The exposure time required for the Disinfectant must not be exceeded and the surface must be washed off immediately with water.

### 3. wash off:

After the exposure time required for the agent, the surface must be cleaned immediately with water and a soft cloth to remove all remaining residues. Some disinfection and cleaners contain additives based on chemical salts. These can stain the surface over time or cause damage after drying if they are not rinsed off with water. We accept no responsibility for such cases.

Caution!

Always ensure correct handling. Failure to do so can lead to brittleness, discolouration and cracking of the upholstery cover within a short time!

---

### 4.3 Disinfectant overview

#### Aldehydes

Disinfectants	Disinfectant manufacturer	Warning
Incidin Extra N	EcoLab	■
Incidin Foam diluted 50% water	EcoLab	■
Kohrsolin Extra N	Bode Chemie	■
Kohrsolin FF	Bode Chemie	■
Lysoform	Lysoform	■
Lysoformin 5%	Lysoform	■
Minutil	Ecolab	■
Nüscosept	Dr. Nüsken Chemie	■
Optisept	Dr. Schumacher	■
Plastisept	Alpro	■

#### Alcohol

Disinfectants	Disinfectant manufacturer	Warning
Bacillol® 30 Foam	Bode Chemie	■
Bacillol® 30 Sensitive Foam	Bode Chemie	■
Bacillol® 30 Sensitive Tissues	Bode Chemie	■
Descosept Sensitive	Dr. Schumacher	■
Incidin Liquid	EcoLab	■
Incidin Pro	EcoLab	■
Mikrozid AF liquid	Schülke & Mayr	■
Mikrozid universal wipes	Schülke & Mayr	■
Pursept A	Schülke & Mayr	■

#### Biguanides

Disinfectants	Disinfectant manufacturer	Warning
Incidin Plus 8%	EcoLab	■
Incidin Plus 8%	Schülke & Mayr	■
AW- Flächendesinfektion	AW-PURE GmbH	■

#### Chlorine, organ. or inorgan. Substances with active chlorine

Disinfectants	Disinfectant manufacturer	Warning
Chloramin-T DAB 9	Laborfachhandel	■
Octenisept	Schülke	■

### Per compounds

Disinfectants	Disinfectant manufacturer	Warning
Dismozone plus	Bode Chemie	■
Dismozone pur 4%	Bode Chemie	■
Incidin Active	EcoLab	■
Incidin oxy Wipe	EcoLab	■
Incidin oxy Wipe S	EcoLab	■
Mikrozid PAA wipes	Schülke & Mayr	■
Wofasteril	Kesla Pharma	■
Wofasteril – Kombiverfahren Wofasteril und alcapur	Kesla Pharma	■
1 + 1 Wofasteril SC super – Kombiverfahren Wofasteril und alcapur	Kesla Pharma	■

### Quaternary ammonium compounds

Disinfectants	Disinfectant manufacturer	Warning
Cleansept Wipes Forte Maxi	Dr. Schumacher	■
Descosept Sensitive Wipes	Dr. Schumacher	■
Incidin Pro	EcoLab	■
Kohrsolin FF	Bode Chemie	■
Mikrobac Tissues	Bode Chemie	■
Pursept A	Schülke & Mayr	■
FD 366 / 333 / 360	Dürr	■
Mikrozid sensitive liquid	Schülke	■

### Warnings:

- good material compatibility
- can lead to changes in the surface
- leads to long-term changes to the surface

### Important notes:

1. the surface must be cleaned before disinfecting.
2. the surface must be rinsed with water after disinfection.
3. Dilute the disinfectant if possible.
4. Do not use microfibre cloths.
5. pay particular attention to points 2 + 3 for disinfectants with the warning ■+■.

If a disinfectant is not listed, it must first be tested in the desired dosage and duration of use at a suitable location!

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When it comes to furnishing and equipping hospitals, surgeries and nursing homes, we are your competent partner. From standard to specific customer solutions, you get everything from a single source.

Consulting / Development / Sales / Service

**HESS Medizintechnik AG**

Grabenstrasse 14  
CH-8865 Bilten  
T +41 55 619 20 80  
info@hess-med.ch  
hess-med.ch